# FarmaMed

**ELETTROSTIMOLATORE PER IL COLLO**  
**NECK ELECTRONIC STIMULATOR**  
**ELECTRO ESTIMULADOR CERVICAL**  
**ÉLECTROSTIMULATEUR POUR LE COU**  
**ELEKTRISCHER MUSKELSTIMULATOR**  
**FÜR DEN NACKEN**  
**ELETROESTIMULADOR PARA O PESCOÇO**

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IT

INSTRUCTIONS FOR USE

EN

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ES

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FR

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DE

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PT

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## Dispositivo Medico.

## NECK ELECTRONIC STIMULATOR

Dear customer, **Farmamed** wishes to thank you  
for choosing this product.

Our name is synonymous with reliable, high-quality products.

For further information on this or other **Farmamed** products,  
visit our website **[www.farmamed.it](http://www.farmamed.it)**.

### INDEX

#### PAG

- 18 \_ Package contents
- 18 \_ Description of the appliance
- 20 \_ Symbol key
- 20 \_ Security warning
- 23 \_ Warnings on battery use
- 23 \_ Warnings on electromagnetic compatibility
- 24 \_ Appliance preparation
- 25 \_ Instruction for use
- 27 \_ Remote control battery replacement
- 27 \_ Maintenance, upkeep and cleaning
- 28 \_ Disposal
- 28 \_ After-sales service and warranty
- 29 \_ Trouble-shooting
- 30 \_ Product specifications
- 87 \_ Appendix

## PACKAGE CONTENTS

- Neck massager (lithium battery provided)
- Remote control (replaceable batteries provided)
- Power cord
- These instructions for use

### Appliance functions

This device is indicated for use by adults for the treatment of neck dystonia, to reduce heavy head feelings and neck pain. This neck massager generates an energy field that passes through the tissues and provides a convenient and practical aid for the treatment of cervical and vertebral conditions in the comfort of the user's own home.


The appliance is intended for personal use only and must not be used for medical or commercial purposes.

### Appliance characteristics


- Ergonomic design, suited to the natural curve of the neck.
- Simple, user-friendly menu of the remote control.
- A range of functions in a single device: infrared hot plate, low-frequency electrical impulses, massage.
- Possibility of selecting the desired temperature level and electrical impulse intensity to suit individual requirements and preferences.
- Two automatic programmes and four programmes that can be set manually.
- Remote control handset with LED display.

## DESCRIPTION OF THE APPLIANCE


**Fig. 1: main unit**

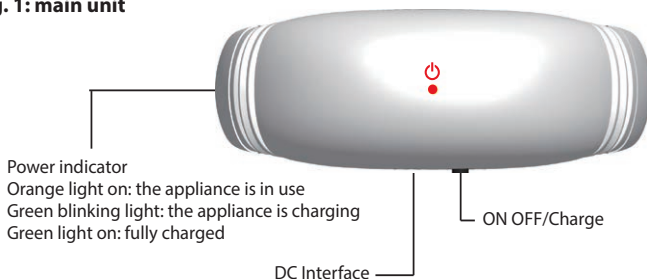
**Fig. 2: main unit, inside view**

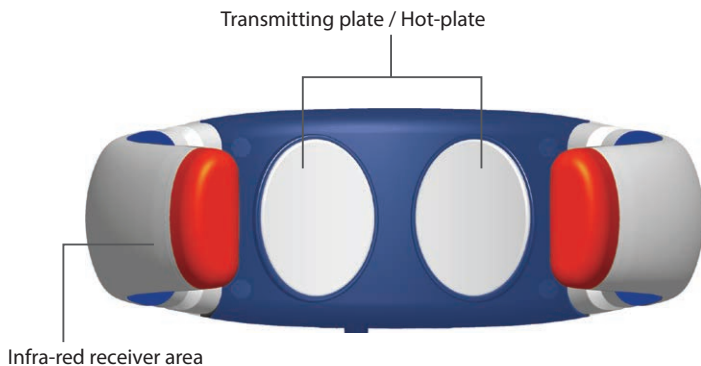**Fig. 3: remote control**

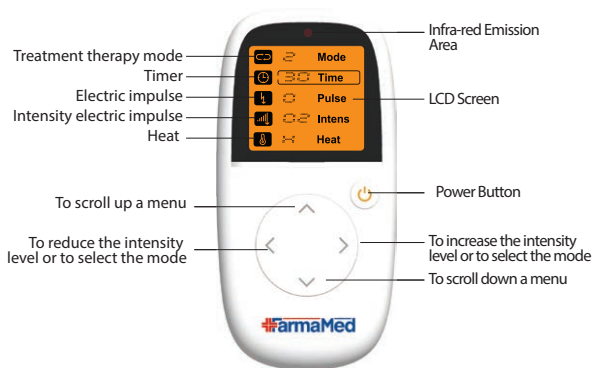**Fig. 4: power cord**
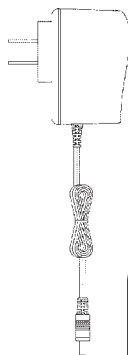
## SYMBOL KEY

The following symbols appear in the instructions for use and on the appliance and its packaging.

**Warning** – Warning regarding possible risk of injury or possible health hazards.

**Note** – Note providing important information.

Dispose of exhausted and completely discharged batteries in the appropriate collection points, at waste collection site for toxic waste or at electronics stores. Battery disposal is a legal requirement.

Disposal in compliance with the provisions set forth in Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).

Serial number.

Please read this manual carefully before using.

According to the degree of protection against direct and indirect contacts, the following equipment is classified as a bf type.

CE label certifies that the device is compliant with the basic requirements set out in the legislation concerning the safety on medical devices.

**IP22** Degree of protection of electrical equipment covers, where the first number indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second number the degree of protection against liquid penetration (from 0 to 8).

## SECURITY WARNING

Before using the appliance, read the instructions for use carefully, paying particular attention to the warnings regarding safety, which must be observed.

This manual must be considered part of the product and must be kept for its entire life-cycle. If the appliance is passed on to another owner, it must be provided together with all the relevant documentation. If the manual is mislaid or for further information or clarification, please contact the manufacturer using the details provided on the last page.

- Before using the product, check that it is intact and has no visible damage. If you have any doubts, do not use the appliance and contact your retailer.
- Using the electrode close to the chest may increase the risk of atrial fibrillation.
- Do not dismantle, repair or tamper with the device. Avoid it.
- If you feel unwell or experience abnormal physical and/or physiological reactions during use, discontinue use of the product and seek medical advice immediately.
- Do not use the appliance on areas close to the heart, head, eyes, front of the neck (in particular, the carotid artery), the lower back, oral cavity or genitalia.
- Do not use on areas affected by skin conditions.
- The device must be switched off and switched back on again before using it, if it is moved or when changing the area being treated.
- If you feel unwell while using the product, discontinue use immediately and seek medical advice.
- When you have finished using the device, disconnect the power cord.
- Do not use together with other electronic medical devices such as pacemakers, artificial heart or lungs and other electronic medical devices with lifesaving systems, ECG machines and other electronic medical devices.
- Do not use the product in places subject to high temperatures, in the presence of flammable substances or in areas subject to electromagnetic radiation and humidity.
- The device must be positioned so that the power cord can be disconnected in an emergency.
- In order to avoid risks, the main unit's battery must be replaced by a professional maintenance technician.
- The skin should not be treated with oily creams or ointments before use.
- Keep the appliance away from heat sources and do not use it in the vicinity (~1 m) of short-wave or microwave appliances (e.g. mobile telephones), in order to avoid undesirable current surges.
- Do not expose the appliance to direct sunlight or high temperatures.
- Protect the appliance from dust, dirt and humidity. Do not immerse the appliance in water or other liquids.
- This appliance is intended for personal use, not for a professional one.
- Remove any metal objects (such as belt buckles or necklaces) that could come into contact with the electrodes during use. Any jewellery or piercings present in the area to be treated (e.g. navel piercing) should be removed before using the appliance.
- Keep the appliance out of the reach of children, do not use it on them unless under constant medical supervision.
- When the device is moved, it must be handled with care, avoiding sudden shaking.
- Pay attention to the device protection, avoiding abrasion.

## Limitations

**The appliance is intended solely for the treatment therapy of the neck and shoulder area of the human body. It must not be considered a substitute for medical treatment. If you are unsure whether use of the appliance is appropriate, contact your doctor.**

### **Use of the device is forbidden in the following cases:**

- patients with pacemakers or other electrical implanted devices and those with suspected or confirmed epilepsy;
- in the presence of wet skin or perspiration or when sleeping;
- patients with brain haemorrhages;
- pregnant and menstruating women;
- in the presence of a haemorrhage following serious injury or a fracture. Seek medical advice;
- subjects with purulent inflammation, blood poisoning;
- in the presence of medical conditions or lesions in the area of the body to be treated;
- subjects with acute cardiovascular or cerebrovascular disease;
- individuals with sensitive skin, heart disease, abnormal blood pressure, malignant tumours, cerebrovascular disease, acute medical conditions or who are undergoing therapy or have other medical conditions;
- children or individuals unable to use the device themselves.

### **Consult your doctor before using the device in the following cases:**

- in the presence of a serious medical condition or after upper-body surgery or subjects who suffer from thrombosis or diabetes;
- in the presence of chronic pain of unknown origin, regardless of the area of the body;
- in the presence of acute medical conditions, especially in the case of suspected or confirmed diseases associated with hypertension, blood clotting disorders, history of thromboembolic disease or cancer;
- in the presence of any kind of skin disease or open wounds;
- in the presence of swelling, skin infections, skin disease or phlebitis;
- in the presence of skin sensitivity disorders or in those who are not sensitive to heat or electrical stimulation applied to the skin;
- on-going medical treatment;
- disorders that present during stimulation treatment;
- skin irritation caused by prolonged stimulation in the same point;
- pregnant and menstruating women;
- individuals with sensitive skin, heart disease, abnormal blood pressure, malignant tumours, cerebrovascular disease, acute medical conditions or who are undergoing therapy or have other medical conditions;
- the product is made of non-surgical quality stainless steel in parts intended to come into direct contact with the skin. In case of doubts about the possible onset of sensitization or allergies, consult your doctor before using the device. **Stainless steel plates contain a small amount of nickel.**

## WARNINGS ON BATTERY USE

- The batteries inserted in the main unit are rechargeable, using the supplied adaptor. They are not replaceable by the user, who must contact Farmamed.
- Do NOT incinerate, even when the batteries are exhausted. **Risk of explosion!**
- Only the remote control uses a disposable battery that can be replaced by the user and is not rechargeable when exhausted.
- Use batteries of the same type and replace them at the same time.
- Batteries must be removed or replaced by an adult.
- Keep the batteries out of the reach of children: swallowing batteries can be fatal. If swallowed, seek medical advice immediately.
- The acid inside the batteries is corrosive. Avoid contact with skin, eyes and clothing.
- If the battery liquid comes into contact with the skin and eyes, wash the affected area with water and seek medical advice.
- In the case of battery liquid leakage, use protective gloves and clean the battery compartment with a dry cloth. Dispose of the materials used for cleaning.

## WARNINGS ON ELECTROMAGNETIC COMPATIBILITY

- The appliance is suitable for use in any condition indicated in these instructions for use, including domestic environments.
- Under certain circumstances, in the presence of electromagnetic disturbance, use of the appliance may be subject to limitations. Consequently, error messages may appear on the display/appliance/remote control.
- Do not use this appliance in the immediate vicinity of other electrical appliances, as this could cause a malfunction.
- Failure to comply with these notes could be detrimental to the performance of the appliance.
- Mobile telephones or appliances that generate electromagnetic fields must not be used in the vicinity of the product.
- Do NOT use the product in environments subject to electromagnetic interference.

Additional quantitative data regarding electromagnetic compatibility are available from Farmamed and can be obtained upon request.

## APPLIANCE PREPARATION

### • USING THE POWER CORD

Connect the adaptor to the device by inserting the plug on the adaptor into the socket on the main unit, then insert the power cord plug into the power socket (Fig. 5).

### • BUILT-IN RECHARGEABLE BATTERY

The device is provided with a built-in rechargeable battery, not user-accessible, for use and operation when the appliance is not connected to an external power source.

1) During use of the device, if the built-in lithium battery level is inadequate, a "DI" tone will sound 3 times. The battery can be charged using the power cord provided, by turning the switch to "charge". The green indicator light on the power cord will flash. Once the battery is fully charged, the green indicator light will remain on. Charging time is approximately 3/4 hours.

2) When fully charged, the device can be used for approximately one week (considering 30 minutes' use a day).

3) The duration of the rechargeable batteries decreases when they are not used for long periods; it is advisable to recharge them fully once a month, after a complete discharge of the battery.

### • REMOTE CONTROL BATTERY INSTALLATION

The remote control is powered by a CR2032 battery (provided).

To insert the battery, open the battery compartment (Fig. 6).

Insert the battery as indicated. Close the battery compartment.

Fig. 5

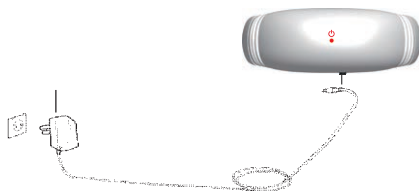
Fig. 6

# INSTRUCTIONS FOR USE

## *Introduction*

For best results, do not use the device more than twice a day, electrostimulation not more than 10~15 minutes each time and heat not more than 15~30 minutes each time.

## *Preliminary steps*

Wipe the neck area and the conductive plate of the device with a damp cloth. Apply the device to the neck, with the device switched off and disconnected. Dry any perspiration on the neck area.

Press the "ON" key on the remote control: the main unit power indicator light and the LED on the remote control will come on. The device is now ready for use and will be set to automatic mode, i.e.:

- high temperature
- level I electrical impulse intensity
- 15-minute session

To discontinue use at any time, press the "ON" key.

The message "DI" may appear on the display when using the remote control. This occurs when the remote control does not receive the device's infrared signal correctly. If this happens, switch the remote control and the main unit off and repeat the previous steps.

## *Description of wireless remote control and device functionality*

The device operates with 4 modes selectable via the supplied wireless remote control :

"^" to scroll up a menu.

"v" to scroll down a menu.

"<" to reduce the intensity level or the mode selected.

">" to increase the intensity level or the mode selected.

### • **Treatment therapy mode "MODE" ( ).**

Select "  " using the "^" and "v" key, press "<" and ">" to select one of the 4 mode options ("MODE"):

**1. Automatic mode:** combines 3 available modes, repeating them in cycles (heat and electrostimulation techniques, heat mode, electrostimulation mode).

**2. Heat + electrostimulation mode.**

**3. Heat mode.**

**4. Electrostimulation mode.**

After selecting the mode, it is possible to set the other parameters using the supplied wireless remote control:

### • Time setting "TIME" ( ).

Select "  " using the " ^ " and " v " key, press " < " and " > " keys to select the time range. the time range can be set to between **5 and 30 minutes** (with 5-minute intervals).

### • Electrical impulse mode "PULSE" ( ).

Select "  " using the " ^ " and " v " key, press " < " and " > " keys to select the electrostimulation mode. There are **6 electrostimulation modes**.

- Mode 1 and 2: **automatic therapies** combine modes 3, 4, 5 and 6.
- Mode 3: it is **scraping therapy**. It is helpful to improve blood circulation.
- Mode 4: it is **massage therapy**. It gives general well-being.
- Mode 5: it is **kneading therapy**. It is helpful to relieve neck.
- Mode 6: it is **thump massage**. It is helpful to accelerates blood circulation, removes toxins, relaxes tense muscles, relieves pain and eliminates fatigue.

### • Electrical impulse intensity "LEVEL" ( ).

Select "  " using the " ^ " and " v " key, press " < " and " > " to increase or decrease electrical impulse intensity. There are **16 electrostimulation intensity levels**.

### • Heat level "HEAT" ( ).

Select "  " using the " ^ " and " v " key, press " < " and " > " to select the heat level desired choosing from the options:

"H": high.

"L": low.

"O": no heat.

## HOW TO BENEFIT FROM ELECTROSTIMULATION

The technique of cervical electrostimulation, if used in a constant and controlled way, allows to increase the cervical-vertebral muscles strength in the medium-long term.

Modes 1, 2, 4 of the Farmamed Neck Electronic Stimulator use electrostimulation therapy alone (MODE 4) or in combination with heat (MODE 1 - MODE 2). You can set both the mode of the electrical impulse ("PULSE") and its intensity ("LEVEL").

After selecting the "MODE" and "PULSE" values, you can set the intensity of the electrical impulse ("LEVEL"), increasing the intensity gradually and carefully.

**Do not use the electrostimulation more than twice a day and not more than 10~15 minutes each time.**

## HOW TO BENEFIT FROM HEAT THERAPY

Heat has always been used as a therapy to relieve muscle and joint pain.

Mode 3 of the Farmamed Neck Electronic Stimulator uses heat to reduce neck tension. With this program it is not possible to set the mode of the electrical impulse ("PULSE") and its intensity ("LEVEL").

**Do not use the heat more than twice a day and not more than 10~15 minutes each time.**

## REMOTE CONTROL BATTERY REPLACEMENT

1. Open the battery compartment as indicated in figure 6.
2. Remove the exhausted batteries and dispose of them as indicated in the "Disposal" chapter.
3. Insert batteries of the same type and with the same service life, making sure they are positioned as indicated.
4. Dispose of the exhausted batteries in compliance with applicable local regulations.
5. Close the battery compartment.

- The remote control is powered by a CR2032 battery.
- When the battery level is low, the brightness of the display is dimmed and the infrared signal does not reach the main unit correctly.
- If the product is not to be used for a long period, remove the battery, which could otherwise leak and damage the device.
- Do not reverse the positive and negative poles of the battery; the positive pole must face upwards.
- Dispose of exhausted batteries in compliance with applicable local regulations. See the "Disposal" chapter of this manual.

## MAINTENANCE , UPKEEP AND CLEANING

Before performing any cleaning operations, switch the device off and remove the plug from the power socket.

- After each use, wipe the conductive plate with a damp soft cloth and dry.
- If the conductive plate is particularly dirty, clean with a soft cloth wetted with a small amount of rubbing alcohol (75% solution).
- Wipe with a damp soft cloth or towel and dry.
- The device must be kept in a dry, well-aired and safe place.
- The device must be transported with care, avoiding excessive shaking.
- It should be adequately protected to avoid scratching.

Never perform cleaning operations when the device is switched on and/or connected.

## DISPOSAL

The product packaging is made of recyclable materials. It must be disposed of in compliance with local regulations.

Dispose of exhausted and completely discharged batteries in the appropriate collection points, at waste collection site for toxic waste or at electronics stores. Battery disposal is a legal requirement.

For environmental reasons, the appliance must not be discarded with normal household waste at the end of its service life. It must be taken to an appropriate disposal facility. Dispose of the appliance in accordance with European Directive 2012/19/EU regarding waste electrical and electronic equipment. For further information or clarification, contact the local authority responsible for waste disposal.

## AFTER-SALES SERVICE AND WARRANTY

**This appliance is intended for domestic use only and is not suitable for professional purposes.**

- The appliance is covered by a two-year warranty from the time of purchase, which must be proven by proof of purchase. This period complies with applicable regulations ("Consumer Code" Italian Legislative Decree no. 206 of 06/09/2005) and only applies if the consumer is a private subject.
- The warranty covers production defects only and is not valid for damage caused by accidents, incorrect use, negligence or improper use of the product. It does not cover materials subject to normal wear and tear.
- The appliance must not be opened under any circumstance; the warranty is not valid if the product has been opened or tampered with.
- The warranty expires 2 years after purchase; after this time, any technical assistance will be provided on a paid basis. For information on technical assistance under warranty or provided on a paid basis, contact Farmamed using the details on the last page of this manual. Consumers will not receive any reimbursement for product repairs or replacements performed under warranty. In the event of a malfunction, contact the manufacturer; Do NOT send anything directly to Farmamed, **send an e-mail to [service@farmamed.it](mailto:service@farmamed.it)** first. Work performed under warranty (including replacement of all or part of the product) will not extend the duration of the original warranty of the replaced product.

- The manufacturer declines all responsibility for any damage/injury that may be directly or indirectly caused to people, property or domestic animals as a consequence of failure to comply with the indications provided in this instruction manual, in particular the warnings regarding the installation, use and maintenance of the appliance.
- **Given its on-going commitment to improving its products, Farmamed reserves the right to modify all or parts of its products in order to meet production requirements, without giving advanced notice, without this entailing any responsibility for Farmamed or its dealers.**

## TROUBLE - SHOOTING

In the event of problems during use of the appliance, consult the information below.

| Problem                                                                           | Possible cause                                                                                               | Solution                                                                                                                               |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------|
| The device will not switch on.                                                    | The switch on the main unit is off.                                                                          | Check that the switch on the main unit is in the "ON" position.                                                                        |
|                                                                                   | The built-in battery level is very low.                                                                      | Recharge the battery using the external power cord.                                                                                    |
|                                                                                   | The device is connected to the power cord, but the built-in battery level is very low.                       | Check that the device is connected to the power cord correctly and that the power cord is working correctly.                           |
| The device cannot be recharged.                                                   | The adaptor is not connected correctly.                                                                      | Check that the power cord is connected to the device correctly and does not present any anomalies (e.g. breaks or kinks in the cable). |
|                                                                                   | The main unit switch is not in the "charge" position.                                                        | Check that the main unit switch is in the "charge" position.                                                                           |
| The display is normal, but the appliance does not respond or generate any signal. | The electrode is not in contact with the skin on the neck.                                                   | Reposition the electrode so that it comes into contact with the skin on the neck.                                                      |
|                                                                                   | The parts of the body that come into contact with the electrode are covered by residues that hinder contact. | Uncover the parts that come into contact with the electrode.                                                                           |

# PRODUCT SPECIFICATIONS

## Product power supply requirements

- Main unit power supply: 100-240 V AC, 50/60 Hz 0.2A frequency converted into 6 V 800 mA DC, by the power cord provided.
- Internal electrical power supply: 3.7 V DC. 1450 mAh
- Remote control power supply: 3V DC from CR2032 battery
- Adaptor specifications
- Input: 100-240 V AC, 50/60 Hz 0.2 A frequency
- Output: 6 V 800 mA DC
- Manufacturer: Chousen International (Shenzhen) Co., Ltd

## Lithium battery specifications:

- Nominal capacity: 1450 mAh
- Nominal voltage: 3.7 V DC
- Dimensions: 8.6x33x43 mm
- Weight: approx. 25 g

## Description of the product's main technical parameters

- Pulse frequency: 1 Hz~400 Hz.
- Pulse width: 20  $\mu$ s~200  $\mu$ s.
- Pulse duration:  $\leq$ 3 seconds.
- Peak output energy per pulse:  $\leq$ 300 mJ.
- Actual value at peak output amplitude:  $\leq$ 25 V 50 mA.
- Device timer: 15 minutes, }10% tolerance.
- Safety classification: internal electrical power supply and appliance class type BF
- Maximum dimensions: Main unit (length 180 mm x width 157 mm x height 56 mm)
- Remote control (length 100 mm x width 50 mm x height 19 mm)
- Estimated service life under normal conditions of use:  $\geq$ 3 years.

## Environmental product requirements

- Operating conditions:
  - Temperature: +5 °C- +40 °C;
  - Humidity: relative humidity 15%-93%;
  - Atmospheric pressure: 700 hPa-1060 hPa.
- Storage and transportation conditions:
  - Temperature: -25 °C - +70 °C;
  - Humidity: relative humidity 0%-93%;
  - Atmospheric pressure: 700 hPa-1060 hPa.
- Transportation conditions:
  - Temperature: -10 °C - +40 °C;
  - Humidity: relative humidity 15%-93%;
  - Atmospheric pressure: 700 hPa-1060 hPa.

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# APPENDICE - APPENDIX - APÉNDICE

## ANNEXE - ANHANG - APÊNDICE

### Guidance and manufacturer's declaration – electromagnetic emissions

This device is compliance for each EMISSIONS test specified by the standard, e.g. EMISSIONS class and group.

| Emissions                                              | Compliance | Electromagnetic environment – guidance                                                                                                                                                                                   |
|--------------------------------------------------------|------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| RF emissions CISPR 11                                  | Group 1    | This device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| RF emission CISPR 11                                   | Class B    |                                                                                                                                                                                                                          |
| Harmonic emissions IEC 61000-3-2                       | Class A    |                                                                                                                                                                                                                          |
| Voltage fluctuations / flicker emissions IEC 61000-3-3 | Complies   |                                                                                                                                                                                                                          |

### Guidance and manufacturer's declaration – electromagnetic immunity

This device is compliance for each EMISSIONS test specified by the standard, e.g. EMISSIONS class and group

| Immunity test                                                                                       | IEC 60601-1-2 test level                                                                                | Compliance level                                                                                        |
|-----------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------|
| Electrostatic discharge (ESD) IEC 61000-4-2                                                         | ±8 kV contact<br>±2 kV, ±4 kV, ±8 kV,<br>±15 kV air                                                     | ±8 kV contact<br>±2 kV, ±4 kV, ±8 kV,<br>±15 kV air                                                     |
| Radiated RF EM fields IEC 61000-4-3                                                                 | 10 V/m<br>80MHz-2.7GHz<br>80% AM at 1kHz                                                                | 10 V/m<br>0.15MHz-2.7GHz<br>80% AM at 1kHz                                                              |
| Electrical fast transient/burst IEC 61000-4-4                                                       | ±2 kV<br>100 kHz repetition frequency                                                                   | ±2 kV<br>100 kHz repetition frequency                                                                   |
| Surge IEC 61000-4-5                                                                                 | ±0.5 kV, ±1 kV line-to-line;<br>±0.5 kV, ±1 kV e ±2 kV<br>line-to-ground;                               | ±0.5 kV, ±1 kV line-to-line;<br>±0.5 kV, ±1 kV e ±2 kV<br>line-to-ground;                               |
| Conducted disturbances induced by RF fields IEC 61000-4-6                                           | 3V<br>0.15MHz-80MHz<br>6V in ISM and amateur radio<br>bands between 0.15MHz and<br>80MHz 80% AM at 1kHz | 3V<br>0.15MHz-80MHz<br>6V in ISM and amateur radio<br>bands between 0.15MHz and<br>80MHz 80% AM at 1kHz |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | 0% U <sub>r</sub> : 0.5 cycle <sup>a)</sup><br>At 0°, 45°, 90°, 135°, 180°,<br>225°, 270°, and 315°.    | 0% U <sub>r</sub> : 0.5 cycle <sup>a)</sup><br>At 0°, 45°, 90°, 135°, 180°,<br>225°, 270°, and 315°.    |
|                                                                                                     | 0% U <sub>r</sub> : 1 cycle<br>70% U <sub>r</sub> : 25/30 cycles <sup>b)</sup><br>Single phase: at 0°.  | 0% U <sub>r</sub> : 1 cycle<br>70% U <sub>r</sub> : 25/30 cycles <sup>b)</sup><br>Single phase: at 0°.  |
|                                                                                                     | 0% U <sub>r</sub> : 250/300 cycles <sup>b)</sup>                                                        | 0% U <sub>r</sub> : 250/300 cycles <sup>b)</sup>                                                        |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8                                             | 30 A/m<br>50 Hz o 60 Hz                                                                                 | 30 A/m<br>50 Hz o 60 Hz                                                                                 |

NOTE

a) U<sub>r</sub> is the a.c. mains voltage prior to application of the test level;

b) E.g. 25/30 means 25 periods at 50 Hz or 30 periods at 60 Hz.

# Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment

| Test Frequency (MHz)                                                                                                                                                                                                                                                                                                       | Band (MHz) | Service <sup>a)</sup>                                                 | Modulation <sup>b)</sup>           | Maximum Power (W) | Distance (m) | IMMUNITY TEST LEVEL (V/m) | Compliance level |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------|-----------------------------------------------------------------------|------------------------------------|-------------------|--------------|---------------------------|------------------|
| 385                                                                                                                                                                                                                                                                                                                        | 380-390    | TETRA 400                                                             | Pulse Modulation 18 Hz             | 1.8               | 0.3          | 27                        | 27               |
| 450                                                                                                                                                                                                                                                                                                                        | 430-470    | GMRS 460<br>FRS 460                                                   | FM $\pm$ 5kHz deviation 1 kHz sine | 2                 | 0.3          | 28                        | 28               |
| 710<br>745<br>780                                                                                                                                                                                                                                                                                                          | 704-787    | LTE Band13, 17                                                        | Pulse Modulation 217 Hz            | 0.2               | 0.3          | 9                         | 9                |
| 810<br>870<br>930                                                                                                                                                                                                                                                                                                          | 800-960    | GSM 800/900,<br>TETRA 800,<br>GSM 820,<br>CDMA 850,LTE Band 5         | Pulse Modulation 18 Hz             | 2                 | 0.3          | 28                        | 28               |
| 1720<br>1845<br>1970                                                                                                                                                                                                                                                                                                       | 1700-1990  | GSM 1800;<br>CDMA 1900;<br>GSM 1900;<br>DECT; LTE Band 1,3,4,25; UMTS | Pulse Modulation 217 Hz            | 2                 | 0.3          | 28                        | 28               |
| 2450                                                                                                                                                                                                                                                                                                                       | 2400-2570  | Bluetooth,<br>WLAN,802.11 b/g/n,RFID 2450,LTE Band 7                  | Pulse Modulation 217 Hz            | 2                 | 0.3          | 28                        | 28               |
| 5240<br>5500<br>5785                                                                                                                                                                                                                                                                                                       | 5100-5800  | WLAN 802.11a/n                                                        | Pulse Modulation 217 Hz            | 0.2               | 0.3          | 9                         | 9                |
| NOTE: a) For some services, only the uplink frequencies are included.<br>b) The carrier shall be modulated using a 50 % duty cycle square wave signal.<br>c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case. |            |                                                                       |                                    |                   |              |                           |                  |

Shenzhen Pango Medical Electronics Co.

Tel: 86-755-33825988 Fax: 86-755-33825989

Sede principale/Main Site: No.25 1st Industry Zone, Fenghuang Road, Xikeng Village, Henggang Town, Longgang District, Shenzhen, Guangdong China

Sede secondaria/Additional site1: 2-4 Floor, No.5 Shanzhuang Rd., Xikeng Village, Henggang Town, Longgang District, Shenzhen City, Guangdong Province, China

Lotus NL B.V.

Indirizzo/Address: Koningin Julianaplein 10, le Verd, 2595AA,  
The Hague, Netherlands.

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**Farmamed S.r.l.**