**MISURATORE DI PRESSIONE DA POLSO
WRIST BLOOD PRESSURE MONITOR
TENSÍOMETRO DE MUÑECA
TENSIONMÈTRE DE POIGNET
HANDGELENK BLUTDRUCKMESSGERÄT
BLODTRYCKSMÄTARE FÖR HANDELLEN
POLSBLÖEDDRUKMETER
MEDIDOR DE PRESSÃO DE PULSO**

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Schermo: Schermo LCD

Circonferenza del polsino: 13,5-19,5 cm

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Dear customer, **Farmamed** wishes to thank you for choosing this product.

Our name is synonymous with reliable, high-quality products.

For further information on this or other **Farmamed** products, visit our website **www.farmamed.it**.

WRIST BLOOD PRESSURE MONITOR

INDEX

PAG

- 27 _ Package contents
- 28 _ Description of the appliance
- 29 _ Symbol key
- 31 _ Security warning
- 33 _ Warnings on battery use
- 34 _ Warnings on
electromagnetic compatibility
- 35 _ General information
- 37 _ Precaution before use
- 37 _ Appliance preparation
- 41 _ Instruction for use
- 43 _ Battery replacement
- 43 _ Maintenance and upkeep
- 44 _ Disposal
- 45 _ After-sales service and warranty
- 47 _ Trouble-shooting
- 48 _ Product specifications

PACKAGE CONTENTS

- Pressure monitor
- Transportation/storage case of the pressure monitor
- These instructions

Device functions

The monitor uses the oscillometric method to measure blood pressure. The automatic electronic blood pressure monitor is designed to be used by medical professionals or at home, to measure and visualise diastolic and systolic blood pressure and pulse frequency, with an air bracelet fastened around the wrist according to the instructions included in "HOW TO CONNECT THE BRACELET". The average life span of this product is 5 years, if used with care and normal frequency. The product is compliant with electromagnetic compatibility requirement EN60601- 1-2, the security standards of EN60601-1 and it guarantees the performances according to IEC 80601-2-30 as required by the EC directive 93/42/CEE.

Characteristics of this device

- Memory up to 180 measurements for 2 people.
- Large and clear LCD Display.
- Visualisation of blood pressure classification.
- Easy to use, automatic measure, measurement values record and the time of measurement with the push of a button.
- Automatic shutdown function within 1 minute of inactivity.

DESCRIPTION OF THE APPLIANCE

Fig. 3: main unit


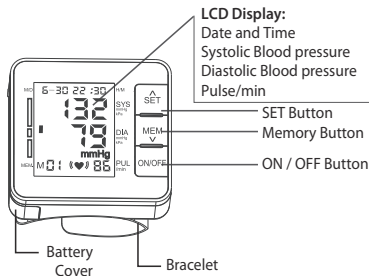
Fig. 4: display

SYMBOL KEY

The following symbols are used in the instructions and on the device and its package.

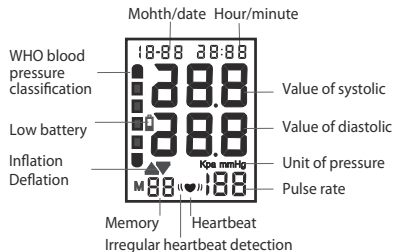Warning – danger of injury or damage to your health.

Note – Note on important information.

Dispose of in compliance with the provisions of Directive 2012/19/EU regarding waste electrical and electronic equipment.

Serial number.

Please read this manual carefully before using.

According to the degree of protection against direct and indirect contacts, the following equipment is classified as a BF type.

Store away from humidity.

Use of the device is not permitted in rooms where a MRI is underway.

CE label certifies that the device is compliant with the basic requirements set out in the legislation concerning the safety on medical devices.

Degree of protection of electrical equipment covers, where the first number indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second number the degree of protection against liquid penetration (from 0 to 8).

SECURITY WARNING

Please read the instructions carefully before using this device, in particular the security warnings, and follow them. This manual must be considered part of the product and it must be conserved throughout the life cycle of the device. Please include the whole documentation when giving the device to a new owner. Please contact Farmamed at the address indicated in the last page if the handbook is lost or if you need further information or clarifications.

- Please check that the device is whole and that there is no visible damage before using this product. In case of doubt do not use the device and contact the distributor.
- The patient must follow the doctor's instructions and must not estimate or give him-herself any treatment based on the measurement's results.
- Self-diagnosis of the measured results and self-treatment are dangerous.
- This device can be used only in domestic or protected environments.
- Press the "ON/OFF" button to interrupt the use when an annoyance to the wrist is perceived or if the bracelet swells anomalously or without pausing.
- Do not allow children below 12 years old and unsound persons to use the device. If used by able-bodied between 12 and 18 years old, they should be monitored by an adult.
- Unsound people should always be monitored by an adult during use, regardless of age.

- Do not use this device for reasons other than blood pressure measurements. The use of this device while moving is forbidden.
- Do not use this device in the open air or in presence of high levels of humidity.
- Do not disassemble, repair or tamper with the main unit or the blood pressure monitor bracelet. The device might no longer work correctly.
- Make sure that the connection tubes are not bent before starting the measurement to avoid physical injuries for the patient or operating errors.
- Do not measure blood pressure more than 3 times at a short distance of time on any patient, wait at least 5 minutes between each measurement.
- Do not measure blood pressure more than 6 times a day.
- Do not use the bracelet on wounds, as it could cause additional injuries.
- Do not measure on the wrist that is on the same side as a mastectomy, as it could cause injuries.
- Read the air pressure value on the LCD display.
- Do not exceed 280 mmHg during measurement, otherwise press the "ON/OFF" button to interrupt.
- Do not apply force when folding the bracelet or the air tube.
- Do not hit or drop the main unit.

- Always use the accessories specified in the handbook, the use of other parts which are not approved by the manufacturer might cause malfunctioning or injuries. In case of doubt, consult Farmamed.
- Please contact the distributor for further information about support, components list, etc.
- Do not carry out support and maintenance interventions while using the device.
- Do not use the equipment in case of contact with water. Consult your dealer before re-use.

WARNINGS ON BATTERY USE

- If the appliance is not to be used for a long period, remove the batteries and store in a cool, dry place at room temperature.
- DO NOT recharge batteries unless they are rechargeable.
- Rechargeable batteries must ONLY be recharged as indicated in the manual provided with them and using the appliances indicated by the manufacturer.
- NEVER expose batteries to sources of heat and direct sunlight. Failure to observe this precaution may damage the batteries and/or cause them to explode.
- Do NOT incinerate, even when the batteries are exhausted. **Risk of explosion!**
- Use batteries of the same type and replace them at the same time.
- Batteries must be removed or replaced by an adult.
- Keep the batteries out of the reach of children: swallowing batteries can be fatal. If swallowed, seek medical advice immediately.
- The acid inside the batteries is corrosive. Avoid contact with skin, eyes and clothing.
- If the battery liquid comes into contact with the skin and eyes, wash the affected area with water and seek medical advice.

- In the case of battery liquid leakage, use protective gloves and clean the battery compartment with a dry cloth. Dispose of the materials used for cleaning.

WARNINGS ON ELECTROMAGNETIC COMPATIBILITY

- The appliance is suitable for use in any environment indicated in these instructions for use, including domestic environments.
- Under certain circumstances, in the presence of electromagnetic disturbance, use of the appliance may be subject to limitations. Consequently, error or failure messages may appear on the display/appliance.
- Do not use this appliance in the immediate vicinity of other electrical appliances, as this could cause a malfunction.
- Failure to comply with these notes could be detrimental to the performance of the appliance.
- Mobile telephones or appliances that generate electromagnetic fields must not be used in the vicinity of the product.
- Do NOT use the product in environments subject to electromagnetic interference.

Additional quantitative data regarding electromagnetic compatibility are available from Farmamed and can be obtained upon request.

GENERAL INFORMATION

1. What is blood pressure?

Blood pressure is the force exerted by the blood against the artery walls. **Systolic pressure** occurs when the heart contracts. **Diastolic pressure** occurs when the heart expands. Blood pressure is measured in millimetres of mercury (mmHg). The natural blood pressure is represented by pressure, which is measured in the early morning while the body is still resting and before eating.

2. What is hypertension and how can it be measured?

Hypertension is an arterial pressure that is abnormally elevated, if not cured, it could cause many health problems including ictus and infarct. Hypertension can be maintained by modifying your lifestyle, avoiding stress and following the doctor's prescriptions.

To prevent hypertension or keep it under control:

- do not smoke;
- reduce your intake of salt and fat;
- keep your weight at an adequate level;
- exercise regularly;
- carry out regular check-ups.

3. Why measure your blood pressure at home?

Blood pressure measured in a clinic or a surgery might cause apprehension and produce a higher reading, that could be of 25 to 30 mmHg higher than one measured at home. The measurement at home reduces the effects of external influences on pressure readings, it integrates the doctor's readings, which are always those to be taken as a reference, and supplies a blood pressure analysis.


4. OMS blood pressure classification:

The standards for the evaluation of high blood pressure, independently from age, have been established by the World Health Organisation (OMS) and recorded in fig.1.

5. Blood pressure variations


An individual's blood pressure varies considerably depending on the days and the season. It can go from 30 to 50 mmHg within a single day. Variations are even more pronounced in subjects who suffer from hypertension. Normally blood pressure increases when one is working or playing and goes down to its lowest level when one is sleeping (fig.2). Measurements must be made at the same time every day, following the procedure outlined in this manual. Many readings give a more complete anamnesis of one's blood pressure. Make sure you take down the date and hour of the measurement of your blood pressure and consult your doctor for correct interpretation of your blood pressure, even when measured independently.

Fig. 1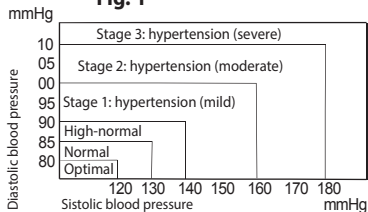

(Extract from: Journal of Hypertension 1999, Vol 17 No.2)

Fig. 2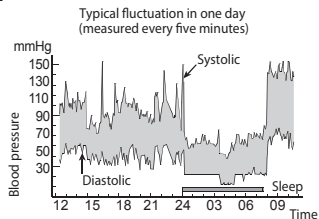

PRECAUTIONS BEFORE USE


- Consult your doctor in order to determine the more appropriate moment to measure your blood pressure when taking medication.
- For persons with problems of irregular or unstable peripheral circulation due to diabetes, liver diseases, arteriosclerosis, etc., there may be fluctuations of the blood pressure values measured in the upper part of the arm compared to those measured on the wrist. Please always consult a doctor to get an accurate clinical picture.
- Measurements can be compromised if this device is used close to televisions, microwave ovens, X rays, cell phone equipment or other devices with powerful electric fields. Please use the device at a sufficient distance or turn off the other devices to avoid this interference.
- Consult your doctor if unexpected readings are obtained.
- The obtained reading is probably slightly lower than the one measured at the hospital because of a more relaxed condition.

APPLIANCE PREPARATION

1) INSERT BATTERIES

- This device works with two 1.5V type AAA alkaline batteries (not included).
- Open the battery compartment to insert the batteries (Fig. 3)

Fig. 5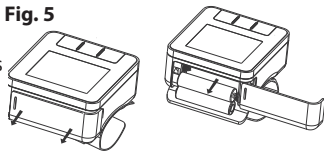

1) INSERT BATTERIES

- This device works with two 1.5V type AAA alkaline batteries (not included).
- Open the battery compartment to insert the batteries (Fig. 3)
- Insert the batteries according to their polarity as indicated. Close the cover.
- Insert new batteries when the symbol () blinks on the display. Do not use new and old batteries at the same time, as this could cause device malfunction and decrease the duration of the batteries.
- The symbol () no longer appears when the batteries are exhausted.

2) TIME SETTING

- Press and hold the "SET" button until the number corresponding to the year appears and flashes on the LCD display to turn on the setting mode.

Fig. 6


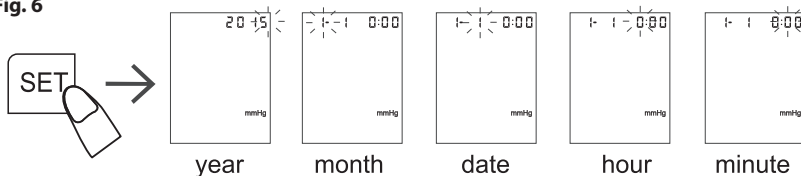
Fig. 7

- Press the “MEM” button to set the year, then push the “SET” button again to save the settings and turn on the month setting.
- Press the “MEM” button to set the month. Follow the same passages to set date/hour/minute.

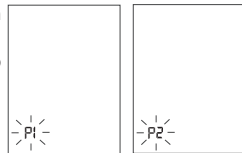3) SELECT USER

When the device is off, press button “SET” to select the user P1 or P2 , then press button “ON/OFF” to measure blood pressure. **Fig. 8**

If you need to inquire memories P1 or P2 average values of user P1 or P2, please firstly press button “SET” to select the user, then press button “MEM” to inquire memories.

4) UNIT CONVERSION

Two blood pressure units are available: mm Hg (mmHg) and kPa (kPa). This device is set for mmHg scale by default.

- Press the “ON/OFF” button for 10 seconds to visualise the interface that allows you to go from one unit to the other.
 - Press the “MEM” button to select **BLOOD PRESSURE CLASSIFICATION**
 - Press the “ON/OFF” button to leave.
- The units will be placed as above between scale mmHg or scale kPa.

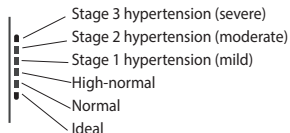
Fig.9: classification of the blood pressure by the world health organisation (OMS)

5) HOW TO PUT ON THE BRACELET

1. Closing the bracelet

- 1) Put the bracelet around your left wrist at approximately 1 to 2 cm below your hand, as shown in the figure.
 - 2) Secure the bracelet using the Velcro band.
- Secure the bracelet on your skin to measure correctly.

2. How to measure correctly

Please follow these instructions for maximum precision of blood pressure measuring:

Sit down comfortably at the table and put your wrist down on the table.

Relax for approximately 5-10 minutes before measuring.


Set your wrist bracelet at the height of your heart.

Don't move or talk while measuring your blood pressure.

Do not measure your blood pressure immediately after exercising or after a bath.

Measure your blood pressure at the same time every day.

Fig. 10

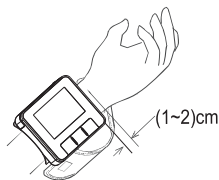INSTRUCTIONS FOR USE

How to measure your blood pressure

1. Put on your wrist bracelet following the instructions described in "How to put on the bracelet".
2. Push the "ON/OFF" button. All the icons will appear on the DISPLAY for two seconds, then it starts to measure, and one can see "0" or the last recorded measure.

Fig. 11

3. When you turn on the monitoring device, the wrist bracelet will start to inflate automatically. The symbol will blink on the LCD display. The results will be displayed at the end.

Fig. 12


Reading the memory

Push the "MEM" button to visualise memorised average values "AUG".

Push the "MEM" button to read the last recorded measures. Use the "MEM" button to scroll down and "SET" to scroll up.

At the end of each measurement it is possible to push the "MEM" button to read the last recorded measure in the memory.

Fig. 13

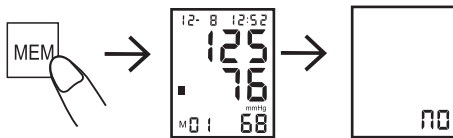Deleting the memory

When you read the memory data, push the memory button for five seconds, the LCD display "no" will show you that all the memory has been deleted.

BATTERY REPLACEMENT

1. Open the battery compartment as indicated in fig.5.
2. Take out the exhausted batteries and dispose of them as indicated in the "Waste disposal" chapter.
3. Insert the same kind of batteries with a similar life span, according to their polarity.
4. Dispose of the exhausted batteries according to local regulations.
5. Close the battery compartment.

MAINTENANCE AND UPKEEP

Follow these instructions to keep your electronic measuring device in the best conditions and protect it from any damage:

- Always store your measuring device in its case when it is unused.
- Do not fold the bracelet too much.
- The textile closing could touch the internal surface of the bracelet and damage it.
- Clean the measuring device and bracelet with a soft dry cloth.
- Do not use abrasive detergents.

- * Do not put the device or its components in water.
- * Do not put the device in extremely high or low temperatures, humidity or direct solar rays.
- * Store the device and its components in a dry and secure place.
- * Do not let the device drop on the floor or any other impact.
- * Remove the batteries if the device will not be used for three months or more. Always change all the batteries at the same time, using batteries of the same type.

DISPOSAL

The product packaging is composed of recyclable materials. Please dispose of it according to local regulations.

For environmental reasons, this device must not be disposed of with normal waste. You must dispose of it in special collection centres. Dispose of this device according to European Regulations Dir.2012/19/UE concerning electric and electronic devices. Please contact local authorities for information or clarifications.

AFTER-SALES SERVICE AND WARRANTY

This appliance is solely intended for domestic use and cannot be used for professional purposes.

- The appliance is covered by a two-year warranty starting on the date of purchase, which must be proven by proof of purchase. This period complies with the national regulations transposing Directives 2011/83/EU, 1999/44/EC, 1999/34/EC on Consumer Guarantees and only applies if the consumer is a private subject.
- The warranty only covers manufacturing defects and is not valid when the damage is accidental or due to incorrect use, negligence or inappropriate use of the product. It does not cover materials subject to natural use and deterioration.
- Never open this device for any reason; the warranty will no longer be valid following the opening or tampering of this device.
- The warranty will automatically expire after 2 years of purchase; following this deadline all support and technical assistance must be paid. Information requests on support and technical assistance, whether during the warranty period or after, must be addressed to the Company address indicated in the last page. The consumer is not required to pay for any repair or substitution during the warranty period. In the event of any defect, please contact the manufacturer; DO NOT send anything directly to Farmamed but **send an email to service@farmamed.it**.
All interventions carried out under warranty (including substitutions of a product or of one of its parts) will not extend the original guarantee of the exchanged product.

- The manufacturer declines all responsibility for any direct or indirect damage to people, things or animals, due to incorrect observation of all the instructions indicated in this booklet, specifically concerning the warnings related to the installation, use and upkeep of this device.
- Farmamed reserves the right to modify its products, completely or partially, without prior notice, according to its production requirements, and this cannot be held against Farmamed or any of their distributors, as they are constantly working to improve their products.

TROUBLE-SHOOTING

Please check the following points if you encounter any issues during the use of the device.

Problem	Possible cause	Solution
E1: can't normally Increase pressure.	Check your wrist cuff if any air leakage.	Contact Farmamed if you need to replace wrist cuff with new one.
E3 inflate pressure too high.	Pressure value of more than 299mmHg.	Contact Farmamed if you need to re-measurement or send back dealer for re-calibrate pressure.
Battery icon on 	Battery low power.	Replace battery and measure again.
The systolic pressure Value or diastolic Pressure value too high.	The wrist cuff was held lower than your heart.	Keeping correct position and gesture to measure again.
	The wrist cuff was not attached properly.	
	You moved your body or s poke during measurement.	
The systolic pressure Value or diastolic Pressure value too low.	The wrist cuff was held higher than your heart.	
	No rest position was observed during measurement.	

PRODUCT SPECIFICATIONS

Product name: Wrist blood pressure monitor

Product dimensions: 72x67x33 mm

Weight: approximately 130 g (excluding the battery)

Measurement range: pressure 30-280 mmHg, pulse 40 - 199 heartbeat/min

Precision: pressure ± 3 mmHg, pulse: $\pm 5\%$

Monitoring method: Oscillometric measurement

Working Conditions:

- **Temperature:** 5.0°C - 40.0°C
- **Maximum relative humidity:** 15%RFI - 93%RFI
- **Atmospheric pressure:** 70kPa - 106kPa

Conditions for storage and transportation:

- **Temperature:** -20°C - 55°C
- **Maximum relative humidity:** 0%RFI - 93%RFI
- **Atmospheric pressure:** 50.0 - 106.0kPa

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Shenzhen Pango Medical Electronics Co., Ltd
Lotus NL B.V.
Farmamed S.r.l. – Via Cisa, 2 - 46034 Cerese di Borgo Virgilio (MN) - Italy