FarmaMed

**PULSOSSIMETRO
PULSE OXIMETER
PULSÓMETRO
OXYMÈTRE DE POULS
PULSOXIMETER
PULSOXIMETER
PULSOXIMETER
OXÍMETRO DE PULSO**

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Display: OLED, colore blu

PULSE OXIMETER

Dear customer, **Farmamed** would like to thank you for choosing this product.

Our brand is a guarantee of high quality and highly reliable products.

For further information on this or other **Farmamed** products, please go to www.farmamed.it.

CONTENTS

PAG

- 19 _ Package contents
- 20 _ Description of the appliance
- 20 _ Symbol key
- 21 _ Safety warnings
- 23 _ Warnings on the use of batteries
- 24 _ Electromagnetic compatibility warnings
- 24 _ General information
- 25 _ Precautions before use
- 26 _ Preparation of the device
- 27 _ Instructions for use
- 30 _ Maintenance and storage
- 31 _ Disposal
- 31 _ Support and warranty
- 32 _ Troubleshooting
- 33 _ Product specifications

PACKAGE CONTENTS

- Pulse oximeter (batteries not included)
- Case
- Carrying strap
- These instructions for use

Functions of the appliance

This device allows non-invasive measurement of arterial oxygen saturation (SpO₂) and heart rate. Oxygen saturation indicates the percentage of oxygen-bound haemoglobin in the arterial blood. It is a vital parameter that allows assessing the respiratory function: a lower than normal oxygen saturation in the blood may indicate a compromised respiratory function and therefore highlight the presence of conditions, such as asthma, heart failure, sleep apnoea, congenital heart defects, etc. In case of a suspected condition, seek medical attention immediately. The device also detects the Perfusion Index (PI), a numerical value that indicates pulse intensity at the point where the sensor is positioned.

Features of the appliance

- Measurement of oxygen saturation in arterial blood (SpO₂)
- Measurement of pulse heart rate (PR)
- Perfusion Index indication (PI)
- Graphical display of heartbeat
- Compact and lightweight, to be used at home or on the move
- Switches on and off automatically
- OLED display

DESCRIPTION OF THE APPLIANCE

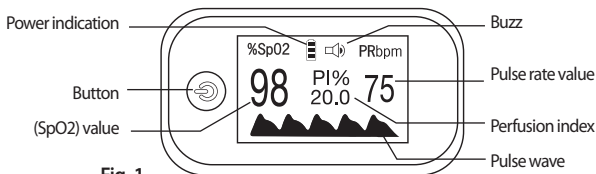
Fig. 1

SYMBOL KEY

The following symbols are used in the instructions for use, on the appliance and its packaging.

Warning – Warning regarding possible risk of injury or possible health hazards.

Note – Note providing important information.

The packaging is made of renewable material and can be recycled.

Serial number.

Please read this manual carefully before using.

Dispose of in compliance with the provisions of Directive 2012/19/EU regarding waste electrical and electronic equipment.

CE label certifies that the device is compliant with the basic requirements set out in the legislation concerning the safety on medical devices.

IP22 Degree of protection of electrical equipment covers, where the first number indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second number the degree of protection against liquid penetration (from 0 to 8).

SAFETY WARNINGS

Before using the appliance, carefully read the instructions for use, especially the safety warnings, and adhere to them. This manual must be considered as being part of the product and must be retained throughout its service life. Should the appliance be handed to another owner, the entire documentation should also be provided. In the event of mislaying the manual or need for more information or clarifications, contact Farmamed at the address provided on the last page.

- Before using the product ensure the appliance is intact without any visible damage. In case of doubt do not use the appliance and contact your dealer.
 - This appliance is solely intended for the use it has been designed for and in the manner indicated in the operating instructions. Any other use shall be considered misuse hence hazardous. The manufacturer shall not be liable for any damage arising from misuse or incorrect use.
 - In the event of fault or malfunction, switch off the appliance without tampering with it. Always contact your dealer for any repairs.
 - Do NOT autonomously disassemble, repair or replace components of this device. Contact your dealer.
 - Keep out of reach of children.
 - The device must not be used by individuals (including children) with reduced physical, perceptive or intellectual capacities or without the necessary experience and/or knowledge, unless under the supervision of a person responsible for their safety or providing them with instructions for using the device.
 - Do not use the device on newborns.
- The device shows a temporary measurement value each time and cannot be used for continuous monitoring.
- Do not perform any self-diagnosis or self-treatment based solely on the result of the measurement without first consulting your general practitioner. Do not take any new medication of your own accord and do not change the type and/or dosage of medicines prescribed for ongoing treatments.
 - The pulse oximeter may not operate with reduced blood flow. Warm up or massage the finger, before taking a measurement.
- The display of this device automatically switches off when there are no readings or poor quality signals.

- In case of prolonged use, personal conditions may require the device to be moved periodically to another finger. Place the device on another finger after checking skin integrity, circulation and proper alignment between the finger and the device.
 - The sensor is motion sensitive. Therefore, keep the finger and the hand still during the measurement.
 - The sensor requires a sufficient blood flow to obtain proper readings. Poor blood flow may cause inaccurate readings. In case of cold hands or poor circulation, it is advisable to warm up the hands to body temperature before taking a measurement.
 - A tourniquet, a blood pressure cuff or other impediments to blood flow may also cause inaccurate readings. Do not measure where they are applied and there is a need to keep them in place.
 - To obtain a correct reading, the finger and the sensor must be cleaned before taking a measurement.
 - Nail polish, French tips or other products on the nails may affect the readings. Do not use such products during a device reading.
 - Make sure the nail on the finger used for the measurement is short enough to allow the fingertip to cover the internal sensors.
 - To avoid situations of discomfort or pain, and for patients with circulation problems, it is recommended not to use the device on the same finger continuously.
 - In order not to interfere with the measurement result, avoid using the pulse oximeter near intense light sources.
 - Clean the device before use.
- Do not sterilise or immerse the device in liquids. Do not spray or spill liquids on the device.
- Do not use caustic or abrasive cleaning agents, or cleaning products containing ammonium chloride to clean the device.
- Protect the device from dust, vibrations, humidity, temperature changes and explosive substances.

There are other conditions that can result in inaccurate readings, including:

- heart rhythm disorders;
- intake of clinical dyes;
- use of arterial catheters;
- thready pulse;
- low levels of haemoglobin in the blood;
- low perfusion;
- high levels of dysfunctional haemoglobin.

WARNINGS ON THE USE OF BATTERIES

- Remove the batteries if the appliance is not used for extended periods of time and store them in a cool and dry place at room temperature.
- DO NOT recharge the batteries if not rechargeable.
- DO NOT recharge the batteries (if rechargeable) with methods other than indicated in the manual provided with them or using equipment not recommended on the battery supplier's documentation.
- NEVER expose the batteries to heat sources and to direct sunlight. Failure to comply with this indication may damage the batteries or cause them to explode.
- DO NOT throw batteries in the fire, even if they are exhausted. **Explosion hazard!**
- Always use batteries of the same type and replace them all at once.
- The batteries must be removed or replaced by adults.
- Keep the batteries out of the reach of children: swallowing the batteries may constitute a death hazard. In the event of swallowing immediately seek medical attention.
- The acid contained in the batteries is corrosive. Avoid contact with skin, eyes or clothing in case of leakage of liquids from the batteries.
- If the battery liquid comes in contact with the skin or eyes, rinse the parts concerned with plenty of water and seek immediate medical attention.
- If the liquid leaks out of the battery, wear protective gloves and clean the battery compartment with a dry cloth. Dispose of the materials used for cleaning.

ELECTROMAGNETIC COMPATIBILITY WARNINGS

- The appliance is suitable for use in any environment listed in these instructions for use, including the domestic environment.
- Avoid using this appliance in the immediate vicinity of other electrical appliances, as this could result in faulty operation.
- Failure to observe these warnings may affect the performance and lifespan of the appliance.
- The use of mobile phones or appliances generating electromagnetic fields in close proximity to the product is not recommended.
- DO NOT use the product in an environment exposed to strong electromagnetic interferences.

Further data on the electromagnetic compatibility of the device are available at Farmamed and can be obtained on request.

GENERAL INFORMATION

This device is a non-invasive instrument which allows measuring and monitoring of blood oxygen saturation level (SpO₂) and heart rate.

Oxygen saturation is expressed as a percentage through a value -SpO₂%- which indicates oxygen saturation in the blood. It is a blood index that makes it possible to determine the degree of respiratory function of the individual. As far as the saturation values are concerned, when these are above 95%, they are to be considered indicatively normal.

The pulse oximeter operates through spectrophotometry.

The device uses a "spring clamp" to grip on the skin, equipped with two probes placed on opposite sides: the first probe generates beams of light in the red and infrared spectrum; these beams pass through the skin and blood flow of the patient and then reach the second probe, which captures the light. In this way, the appliance is

able to measure oxygen saturation in the patient. The amount of light absorbed by haemoglobin is different depending on whether it is more or less saturated. Oxygen-bound haemoglobin (i.e., oxyhaemoglobin - HbO₂) absorbs mainly infrared light; unbound haemoglobin (Hb), on the other hand, absorbs mainly red light. By utilising this difference in absorption between oxygen-bound and unbound haemoglobin, measuring and analysing the difference between the amount of light radiation emitted by the diodes and the final radiation detected by the sensor, the calculation unit incorporated in the device is able to process and in the end provide the oxygen saturation value that will be shown on its display.

PRECAUTIONS BEFORE USE

The device should NOT be used:

- if the appliance or the finger used for the measurement are wet;
- on young children or newborns;
- during MRI or CT scans;
- while measuring blood pressure on the arm with a sphygmomanometer equipped with blood pressure cuff;
- in the presence of nail polish, plasters or dirty fingers;
- on too thick fingers that can only be inserted into the appliance by force;
- on fingers with anatomical alterations, oedemas, scars or burns;
- on too thin or too narrow fingers, e.g. those of young children;
- on patients showing symptoms of agitation at the point of application (e.g. in the presence of tremors);
- in the vicinity of flammable or explosive gas mixtures;
- on fingers with wound dressings (e.g.: plasters, gauzes, etc.)

PREPARATION OF THE DEVICE

1) INSERT BATTERIES

- This device works with two AAA 1.5V replaceable alkaline batteries (not included).
- To insert the batteries open the battery compartment (Fig. 2).
- Insert the batteries respecting the indicated polarity. Close the cover.
- When the "⏻" symbol blinks on the display, quickly replace all batteries with new ones. Do not use old and new batteries at the same time. This may shorten battery life or cause the device to malfunction.

2) APPLY THE STRAP

To comfortably carry the device, a strap to be attached to the appliance is provided in the package. Apply it as follows:

- Thread the thinnest end of the strap through the holder.
- Pass the other end of the strap through the eyelet formed by the narrower end, then tighten (Fig.3).

Fig. 2


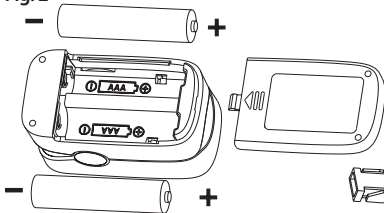
Fig. 3

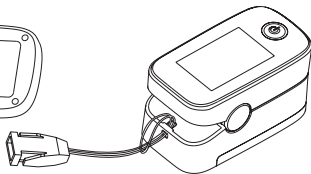INSTRUCTIONS FOR USE

 During the measurement DO NOT look directly at the infrared light emitted by the probe inside the device. The red light and the invisible infrared light from the pulse oximeter are harmful to the eyesight.

1) Switch the device on by briefly pressing the power button.

2) Open the clamp and insert the finger completely inside the opening, with the nail facing upwards. It is recommended not to move the body during the reading and to keep the finger still.

3) Release the clamp, thus closing the sensor on the finger.

4) After a few seconds the display will show the result of the pulse rate and oxygen level measurement (Fig.4).

Note: the pulse rate will not stabilise, but will change constantly because the heart rate is based on heartbeats.

5) When no signal is present, a low signal is detected, or the finger is removed from the device, the unit automatically turns off.

6) Once the finger is removed from the pulse oximeter, the device will automatically switch off after a few seconds.

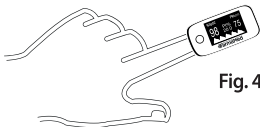
Fig. 4


In order to use the device properly and better interpret the results, it is advisable to take at least 4 measurements on several fingers. The pulse oximeter can be used on the following fingers: right index finger, left index finger, right middle finger, left middle finger. By taking measurements on these fingers, you will obtain 4 values. Out of these, discard the lowest value and add the other three together: divide the sum obtained by three and the number obtained is the one you should take into account and communicate to your doctor, if necessary.

CHANGING THE DATA DISPLAY

With the device switched on, it is possible to change the data display mode and the display orientation by pressing the power button (Fig. 5).

Fig. 5

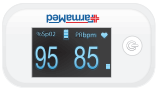VIEW 1 – SUITABLE FOR LEFT-HAND INDEX FINGER	VIEW 2 – SUITABLE FOR LEFT-HAND INDEX FINGER
	
VIEW 3 – SUITABLE FOR RIGHT-HAND INDEX FINGER	VIEW 4 – SUITABLE FOR RIGHT-HAND INDEX FINGER
	
VIEW 5 – VERTICAL STRAIGHT VIEW	VIEW 6 – VERTICAL UPSIDE DOWN VIEW
	

CHANGING THE DEVICE SETTINGS

By holding down the power button for a few seconds, it is possible to access a menu to change some of the device settings. The device has two menus: Sound Setup and Alarm Setup. It is possible to switch from one interface to the other by holding down the power button for a few moments.

SOUND SETUP

Settings		
Alm setup		
Alm	on	*
Beep	off	
Demo	off	
Restore	ok	
Brightness	3	
Exit		

ALARM SETUP

Settings		
Sounds setup		
SpO2 Alm Hi	100	*
SpO2 Alm Lo	94	
PR Alm Hi	130	
PR Alm Lo	50	
+/-		+
Exit		

SOUND SETUP

Press the device power button once to move the "*" symbol to the desired option. Press and hold the power button longer to change the setting.

- When "Alm" is set to "on", and the detected SpO2 or PR values are below the minimum or above the maximum limit, the device will emit an acoustic signal. Instead, when "Alm" is set to "off", and the detected SpO2 or PR values are below the minimum or above the maximum limit, the device will not emit any acoustic signal.
- If "Beep" is set to "on", during the measurement, the device will emit an acoustic signal, synchronised with the pulse. If "Beep" is set to "off", the device will not emit any acoustic signal.
- Move the "*" symbol to "Restore" and hold down the power button to restore the device to factory settings.
- Through the "Brightness" option, it is possible to change the brightness of the screen by selecting a value from 1 to 5: the higher the value, the brighter the display will be.
- Move the "*" symbol to Alm Setup and press the power button to switch to the second menu screen.

ALARM SETUP

Through this screen, it is possible to set the minimum and maximum limits to trigger the alarms related to SpO2 or PR measurement.

- Move the "*" symbol to "+/-". Press the power button and select "+" to increase the minimum and maximum limits, or "-" to decrease them.
- Change the minimum or maximum level of SpO2 or PR by moving the "*" symbol and holding down the power button. The numbers will increase or decrease by pressing the power button once. If "+" was selected in the previous item, the minimum or maximum values will increase. Conversely, if "-" was previously selected, the values will decrease.
- Move "*" to "Exit" and press and hold the power button to return to the display with the measurement data.

MAINTENANCE AND STORAGE

Before and after each use it is recommended to clean the body and the internal silicone surface of the device with a soft cloth, slightly dampened with alcohol. Do not spill or spray liquids on the sensor. Allow the sensor to dry thoroughly before use.

The sensor requires no calibration or maintenance.

It is recommended to store the product in a dry place. A wet environment could compromise the lifespan of the product and even damage it.

DISPOSAL

The product packaging is made of recyclable materials. It must be disposed of in compliance with local regulations.

For environmental reasons, the appliance must not be discarded with normal household waste at the end of its service life. It must be taken to an appropriate disposal facility. Dispose of the appliance in accordance with European Directive 2012/19/EU regarding waste electrical and electronic equipment. For further information or clarification, contact the local authority responsible for waste disposal.

SUPPORT AND WARRANTY

This appliance is solely intended for domestic use and cannot be used for professional purposes.

- The appliance is guaranteed for 2 years from the date of purchase which must be evidenced by proof of purchase. This term complies with the laws in force ("Consumer Code" It. Legislative Decree No. 206 dated 06/09/2005) and is only applicable if the consumer is a private citizen.
- The warranty only covers manufacturing defects and is not valid if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. It does not cover materials subject to natural consumption or wear due to use.
- Do not open the appliance for any reason. In the event of opening or tampering, the warranty shall be forfeited.
- After 2 years from purchase, the warranty expires, and technical support will be provided for a consideration. Information on technical support, be it under warranty or for a consideration, can be obtained by contacting Farmamed at the address provided on the last page. The consumer does not owe any form of consideration for product repairs and replacements that fall under the warranty. In the event of faults contact the manufacturer. DO NOT ship directly to Farmamed but send an mail to [**service@farmamed.it.com**](mailto:service@farmamed.it) first. No warranty servicing – including to replace the product or parts thereof – shall extend the term of the original warranty of the replaced product.
- Farmamed S.r.l. disclaims any liability for any damage that may arise to people, property and animals, either directly or indirectly, as a consequence of failure to comply with all the requirements set out in this instruction manual, especially those concerning the warnings on installation, use and maintenance of the appliance.
- **Since Farmamed constantly strives to improve its products, it is entitled to modify all or part of its products without prior notice in connection with production requirements, without incurring any liability by Farmamed or its dealers.**

TROUBLESHOOTING

In the event of problems experienced while using the unit, please check the following points first.

Problem	Possible cause	Remedy
The device does not turn on.	The battery is low or getting depleted.	Install new batteries.
	The batteries are not installed correctly.	Try inserting the batteries again as described in these instructions for use.
SpO2 and PR values are not detected.	The finger is too small or too big.	The device may not function properly with fingers outside the following size: 8-25.4mm.
	There is too much light in the room.	Try to reduce the excessive ambient light in the room where the measurement is taking place.
	Perfusion index is too low.	Warm up the finger and try to take a new measurement.
The display switches itself off after a few seconds.	No signal detected.	The device is designed to switch off automatically after a few seconds in the absence of detection. Try removing the finger, warm it up and take another measurement.
	The battery is low or getting depleted.	Install new batteries.
The SpO2 value and heartbeats change and do not stabilise on a precise value.	The finger is not inserted correctly.	Remove the finger and try again, making sure that it is inserted deep enough.
	The finger or body are not stationary.	Try to stay still during the measurement.
	The device is used in environmental conditions other than those suggested in this manual.	

If the unit does not work properly after checking the possible causes mentioned above, please contact your dealer.

PRODUCT SPECIFICATIONS

Product description: Pulse oximeter

Dimensions of the product: 57×30×31 mm

Weight: 28 g (without batteries)

Measuring range:

- **SpO₂:** 30-99%
- **Pulse:** 30-250 beats per minute

Accuracy:

- **SpO₂:** 70-100%, ±2 digits, 0-69% no definition

Because pulse oximeter equipment measurements are statistically distributed, only about two-thirds of pulse oximeter equipment measurements can be expected to fall within $\pm$ arms (Average Root Mean Square) of the value measured by a CO-oximeter.

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Shenzhen Yimi Life Technology Co., Ltd
Shangai International Holding Corp. GmbH (Europe)
Farmamed S.r.l. – Via Cisa, 2