

## **Canine Pregnancy Test**



Store at Room Temp

An economical way to determine canine pregnancy.

Accurate results in minutes.

Easy to use.



BioMetallics, Inc. 1-800-999-1961 or 1-609-275-0133 Fax 1-609-275-9485

Manufactured by S&C Biotech

Relaxin is a hormone which is produced by the corpus luteum when the fertilized egg implants. Relaxin will detect the presence of this hormone in the blood, and is a reliable and specific test for pregnancy diagnosis.

The test is best done approximately 30+ days after breeding.

Relaxin™ will also determine a pseudopregnancy since there will be no Relaxin circulating.

Relaxin testing is also important for monitoring pregnancy. Use Relaxin™ to detect any rapid drop in serum relaxin which would indicate a lost pregnancy.

## **Canine Relaxin Procedure**

Collect blood in the small tube enclosed. Allow blood to sit 30 min at room temperature.

Spin or wait until Sera separates from blood.

- **1** Add 2 drops of Sera or Plasma to the Cassette.
- 2 Wait 10-15 minutes. Take Photo.

Two lines with very faint test line is pregnant.

No line is invalid.

